## IT

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10. **ATTENZIONE:** In caso di incidente, please, inform the manufacturer and the local competent authority.

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## EN

## READ CAREFULLY PRIOR TO EACH TOOL USE

**Precautions and Instructions for use.**  
This document is the **essential guide** for a proper and safe use of the **FitEvo Evo and ActivSIT** tool as it contains the **information necessary** so that the product can be used with **user satisfaction** and in compliance with **safety standards** for persons throughout the entire useful life of the product itself and/or its parts.

## INTENDED USES - LIMITS AND EXCLUSIONS

1. This product has been designed and manufactured for the medical field. It should not be used for purposes other than those specified by **Ledragomma**.

## 2. It is not a toy.

## PACKAGE CONTENTS

3. The package contains one tool.

## INFLATION/DEFLATION

4. For proper use, the device should be used with the **Hand Pump 19 cm - 140 CC or 32 cm - 300 CC** to inflate/deflate the tool. Do not inflate by mouth.

## METHODS AND INSTRUCTIONS FOR THE FIRST USE

5. Proceed according to the following instructions for first use.

5.1) The product is equipped with an internal valve suitable for inflation/deflation.

5.2) Position the hand inflator before inserting it in the valve and slowly inflate the product, using pacings.

5.3) Adapt inflation to needs.

## SURFACES TO BE USED

6. The tool should be used on soft, smooth surfaces. Never use the product in the presence of pointed or sharp objects, nor near heat sources, nor leave the unused product under direct exposure to sunlight.

## WARNINGS FOR REPAIRS

7. In case in which there are holes or cuts are present, do not attempt to repair them as any "do-it-yourself" repairs can affect product integrity.

## USABILITY

8. A 4-digit code indicating the month and year of item manufacture is found on the product. Please cite this code in the event of a complaint as well to serve to identify the date and lot of materials used.

## LIMITATION OF LIABILITY

9. Ledragomma S.r.l. declines responsibility for any damage to persons, animals, property or the environment deriving from an use of products not in accordance with the use and maintenance instructions contained in this instructions manual or with warnings provided with the product.

10. **ATTENTION:** In case of an accident, please, inform the manufacturer and the local competent authority.

## DISPOSAL

10.1. The device will have to be disposed of according to the local current law in force.

11. Colors and decorations may vary according to manufacturer.

NOTES:  
11. Colors and decorations may vary according to manufacturer.

Made in Italy by: **Ledragomma S.r.l. - Zona Industriale - Via G. De Simon - 33010 Doppio (UD) - Italy - Tel. +39 0432998649**  
Fax: +39 0432998255 - info@ledragomma.com - info@ledragomma.com - www.tonkey.it

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Yüksek: **Ledragomma S.r.l. - Zona Industriale - Via G. De Simon - 33010 Doppio (UD) - Italy - Tel. +39 0432998649**
Informazioni: **Ledragomma S.r.l. - Zona Industriale - Via G. De Simon - 33010 Doppio (UD) - Italien**
Proizvođač: **Ledragomma S.r.l. - Zona Industriale - Via G. De Simon - 33010 Doppio (UD) - Italy**